## Quick Start Guide

### CA-PX Wireless Vibrating Personal Receiver

#### Installation Charging:

1. Turn Power switch to **OFF** position. Pull out & remove battery tab to engage pre-installed batteries.
2. Plug AC adapter into electrical outlet. Connect adapter plug to CA-PX. Push Power **ON**.
3. Charge time: 3 - 4 hours.

#### Power On/Off:

1. Push Power switch **ON** (towards power icon). Flasher will flash, vibrate & sound a beep.
2. Push Power switch **OFF**. The unit will turn **OFF**.

#### Speaker On/Off:

1. Push Speaker switch **ON**. Unit will beep once.
2. Push Speaker switch **OFF** (away from speaker icon). Unit will vibrate once.

#### Tactile Alert Identification:

1. Each alert has a unique vibration.
2. User presses each button sequentially to find the one that changes the unique vibration pattern into a continuous vibration pattern.
3. User then identifies alert by judging button position or reading Braille on button.

Need Help? Call Toll-free Help Line:  
866-376-9271 (8AM - 5PM PST, M-F)

#### Operations:

1. Turn flashlight **On/Off** - Press **RESET**
2. Send **SOS** for help - Press & hold **SOS** for 2 seconds.
3. Incoming alert - icon button will blink, vibrate & sound (if speaker is On).
4. Acknowledge & stop an alert - Press **RESET**
5. Recall last alert - Press and hold **RESET** for 2 seconds.

#### Learn Accessories (as a Stand-alone Receiver) :

1. Press & hold the **LEARN** button for 6 seconds. **LEARN** button LED will blink for 20 seconds.
2. Press & hold accessory's test button for 10 seconds until CA-PX flashes, vibrates, & sounds (if speaker is On) **Two (2)** times.
3. Repeat steps above for each accessory.

#### Learn CA-PX into a CA-360 System:

1. Press & hold **SOS/TEST** button on CA-360 for 6 seconds until it sounds a beep.
2. Press & hold **LEARN** button on the CA-PX for 10 seconds until CA-360 sounds 2 times & the display shows "**SUB**".
3. Press the **RESET** button on CA-360 to exit.

#### CA-PX Alert Vibration Training:

1. Press and hold **LEARN** button while turning on power.
2. **LEARN** button LED is lit
3. Press any icon button to learn vibration pattern.
4. Press Reset button to stop alert.
5. Press another icon button to learn different vibration.
6. Power off unit to exit vibration training.

Serene Innovations, Inc.  
14731 Carmenita Road, Norwalk, CA 90650  
Local 562.407.5400 www.sereneinnovations.com

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