SERENE[®]
innovations
Making Life Easier

Vibrating Alarm Clock Operating Guide

Model
VA-3

Thank you for selecting the Serene VA-3, vibrating alarm clock. The VA-3's powerful sound and vibration is designed to wake-up even the heaviest sleeper.


BATTERY SAFETY AND DISPOSAL:

Model VA-3 is supplied with three AA batteries. Please do not dispose of used batteries in the household trash. Please take the batteries to a retail or community collection point for recycling. For customers living in the European Union, please take the batteries to a recycle center, or return to factory for proper disposal.

PACKAGE CONTENT CHECK:


VA-3 Vibrating alarm clock

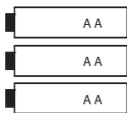Three AA Batteries


Pillow clip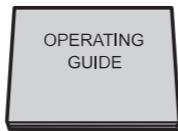

Operating guide

FEATURE IDENTIFICATIONS:

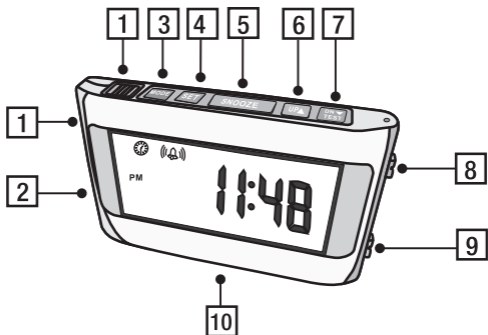1. Built-in flashlight
2. Alarm mode switch (All, shaker, audio, off)
3. MODE (alarm, timer, temperature)
4. SET to select setting choice
5. SNOOZE
6. UP
7. DN/TEST (down or test)
8. Audio alarm tone control (Hi-Med-Lo)
9. Audio alarm volume control (Hi-Med-Lo)
10. Battery compartment (rear)

FEATURE IDENTIFICATIONS (cont.):

Install three AA alkaline batteries (included). Please observe battery polarity while installing. A set of brand new alkaline batteries will last about six to eight months of normal useage.

Please replace batteries if you notice the shaker is weak or if the back-light is dim.

OPERATION:

1. To set time

1. Press and hold SET for 3 seconds.
2. Press UP to display 12 or 24 hour clock. Press SET to select.
3. Press UP or DN to set the hour and to select AM or PM. Press SET to select. Repeat these steps to set Minute.

2. To set alarm time and snooze

1. Set Alarm Mode switch (on the left side) to "ALL" position.
2. Press MODE button to go to Alarm mode. Press and hold SET button for 3 seconds to select.

OPERATION (cont.):

3. Press UP or DN button to set the alarm hour. Press SET to save. Repeat these steps to set Minute.
4. While SNOZ (SNOOZE) is flashing, press UP button to turn SNOZ OFF or ON and then press SET button to save. Press SET button or wait 15 seconds to exit.
5. SNOOZE function (SNOZ is on)---when the alarm sounds off, the alarm will continue for 40 seconds and then stop. If any buttons on the top of the clock are pushed at any time while the alarm is sounding off, the alarm will stop immediately, but will sound off again in eight minutes. You can repeat this cycle for up to seven times. To stop the alarm and snooze completely, press and hold the SNOOZE button for 3 seconds. Your alarm and SNOOZE will reset and your alarm will ring at the same time

OPERATION (cont.):

the next day.

6. If SNOZ (SNOOZE) is set to off, press the SNOOZE button while the alarm sounding off stops the alarm completely.

3. To set alarm on/off & alarm mode

1. Slide the switch to OFF to disable the alarm or to stop the ongoing alarm action.
2. Slide the switch to AUDIO, SHAKER or ALL (audio + shaker) to select your alarm mode and to set the Alarm ON.

4. To test your alarm setting

(alarm setting must be on)

1. Press and hold TEST for 3 seconds. The alarm is triggered for a period of 10 seconds and then returns to the clock mode.

5. To use clock as a Timer

1. Press MODE to go to the Timer mode. Press and hold SET for 3 seconds to select.

OPERATION (cont.):

2. Press UP or DN to set the desired hour setting. Press SET to select. Repeat these steps to set the desired minute setting.
3. Press DN to start/stop the timer countdown (If DN is not pressed within fifteen seconds, clock will return to the regular timekeeping mode).
4. When the set time expires, the alarm will sound off (same alarm mode as the alarm clock setting) for 30 seconds (unless stopped by pressing any of the top keys).

6. To show the room temperature

1. Press MODE to go to the Thermometer mode. The room temperature appears.
2. If needed, press UP to set reading in C or F.
3. Press MODE to return to the clock mode.

SPECIFICATIONS:

- Functions: clock, alarm clock, timer, thermometer and flashlight
- Clock modes: 12 or 24 hours
- Alarm modes: audio, vibration, audio & vibration
- Thermometer modes: C or F
- Size: 118 x 71 x 27 mm; 4.6 x 2.8 x 1.1 inches
- Weight: (without batteries) 140gm; 5 oz.
- Power supply: 3 AA alkaline batteries (provided)
- Battery life: 8-10 months of normal use
- Vibration: 8200+- RPM at .21A
- Flasher brightness: 2000 mcd

WARRANTY SERVICE INFORMATION:

Your **VA-3** comes with one-year limited warranty. We warrant during this warranty period – from the date of purchase to the original consumer – the **VA-3** is free from defects in materials and workmanship under intended home use. In the event that **VA-3** fails to function properly within one year of the original purchase, due to defects in materials or workmanship,

WARRANTY (cont.):

return the unit (freight prepaid) with proof of purchase (sales receipt or packing slip, no exceptions) to Serene Innovations. Serene Innovations will either repair or replace the unit (with a refurbished unit or a unit of equal condition) and return it to you (using UPS/USPS ground shipping) at no cost to you, if the unit is returned within 30 days of purchase. If the unit is returned after 30 days of purchase, but within the warranty period, there will be a warranty handling charge of US \$9.95 for each return. This warranty gives you specific legal rights, and you may also have other legal rights that vary from state to state.

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Copyright © 2018 Serene Innovations, Inc. All rights reserved.
Serene Group, Inc.