USER'S MANUAL Emergency Help Button

English

Model CA-SOS is an optional accessory for the **CentralAlert™** Visual Alert System. It is a wireless personal pager designed for paging HELP.

When activated, a wireless signal is sent to a **CentralAlert™** Master-Unit. This activates all alerting outputs while "SOS" is shown on its display.

Model CA-SOS is a water-resistant design and is supplied with a lanyard. The unit can be mounted inside a shower stall or near a bathtub.

Operating Guide

Model
CA-SOS

Thank you for purchasing this product, please read before use
(Keep this manual for future reference)

2

English

2. This indicates the unit is working normally.

Registering Model SOS:

Model CA-SOS must be learned by a Master-Unit in order to work with the system.

1. Press and hold the "SOS/Test" button on the **CentralAlert™** Master-Unit for about 5 seconds (in **RUN** mode) to set it to a Learn mode. Digital display shows "LEAr."
2. Press and hold the "SOS/Test" button on Model CA-SOS for about 5 seconds until the Master-Unit responds.
3. Registration is now completed. Press the "SNOOZE/RESET" button on the Master-Unit to end the Learn mode.

4

English


Testing:

1. Press the "SOS/Test" button to test.
2. Confirm that the Master-Unit activates all alerting outputs in response.

Installation:

1. Model CA-SOS comes with a lanyard and may be worn as a personal pager. It can also be mounted near any convenient place such as bathroom or kitchen (use a piece of 1" x 1" double sided sponge tape).

English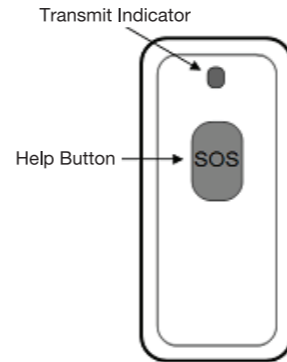


Replace Battery:

1. Press on latch to separate SOS button from back plate.
2. Unscrew screws from corners, separate center section from unit.

6

English

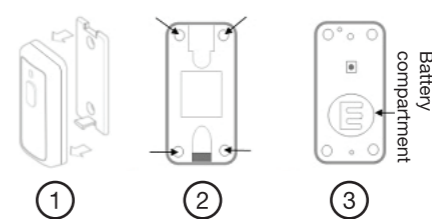3. Change battery - remove the old battery first. Pay attention to the battery orientation - the coin battery is in correct position when the writing on it is face up (towards you, as you look at it).
4. Slide new coin battery all the way into battery compartment; reassemble unit.

Specifications:

1. Advanced Technology: 2.4 Ghz Smart-Code
2. Operating range: Up to 200 ft, open space (outdoors) - actual distance varies according to house construction.
3. Up to four Model CA-SOS' are supported
4. Battery type: size 2032 coin battery
5. Battery life: up to 24 months
6. Operating temperature: +10 to +40°C
7. Size: 2.5" x 3.8" x 1" Wt: 4 Oz
8. Spec is subject to change

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Serene Innovations, Inc.